

LOJER MANUTHERA 242 Cleaning instructions



WARNING

- These instructions do not replace the actual instructions for use. Read the instructions for use before using the device.
- Before starting cleaning, make sure that the power cord is disconnected and that the device's functions have been locked. Check by testing the functions.
- Devices equipped with a battery system can be adjusted even when the power cord is unplugged or during a power outage as long there is power in the batteries. The operations of the device may be prevented by using the locking feature.
- Do not machine wash or use water spray for cleaning. Do not clean in high temperature or air humidity by using e.g. steam or hot water.
- All surfaces must be allowed to dry after cleaning or disinfecting before using the device or its accessories.

↑ CAUTION

- Clean the product as instructed before use.
- Do not use any unsuitable cleaners or disinfectants for cleaning and disinfecting the device. See the instructions below. Follow the respective manufacturer's instructions.
- Allow the surface of electric components to cool off before carrying out maintenance or cleaning procedures.
- Avoid moisture entering the connection points. Excessive moisture can cause liquid pooling and damage the device.



Cleaning

Clean the device aseptically following the work order: From top to bottom and from cleanest to dirtiest. Take into account the following things when cleaning the device:

- The device cannot be machine washed.
- Clean stains and visible dirt as soon as possible.
 - → All secretion stains should be removed immediately after they appear.
 - → Some substances used in care work may cause permanent stains.
- Make sure that the power cord is disconnected and that the device's functions have been locked.
- The surfaces should always be cleaned before disinfection.
 - → Follow the cleaning instructions given by the detergent manufacturers.
- In order to keep the surfaces in good condition, clean the device regularly.
 - → Always clean the device between patients.
- Any surface should not be subject to long-term exposure by any type of liquids.



Frame and moulded soft surfaces

- Clean the **frame and other hard surfaces** with a damp (micro)fiber cloth and a mild detergent solution (neutral pH 6–8 or weakly alkaline pH 8–10). Pay special attention to thorough cleaning of contact surfaces.
- Clean all **moulded soft surfaces** with a damp (micro)fiber cloth and a neutral detergent solution (pH 6–8).
 - → For example, Desiplint (available at www.lojer.com) can be used to clean and disinfect the device.
 - → Do not use e.g. any solvents, abrasive cleanings agents or scouring pads as they can damage the surfaces.
- Use a soft brush to clean difficult stains, corners and other hard to reach places on hard surfaces.
- Remove detergent residues or excess detergent by wiping the surfaces with a cloth dampened with clean water (follow the respective detergent manufacturer's instructions).
- Allow the surfaces to fully dry before using or storing the device.



Disinfecting

The surfaces should always be cleaned before disinfection. Use disinfectant only if justified (e.g. to prevent the transmission of harmful microbes) as disinfection agents might change the surface structure of materials over time.

- All secretion stains should be removed immediately after they appear.
- Follow the disinfecting instructions given by the disinfectant manufacturers.
- Any surface should not be subject to long-term exposure by any type of liquids.

All surfaces

- Disinfect the surfaces with a damp (micro)fiber cloth, using disinfectants suitable for disinfecting medical devices, in accordance with the respective manufacturer's intended purpose and instructions for use.
 - → For example, max. 70% solution of ethanol or propanol can be used to disinfect secretion stains.
- Wipe the surfaces with a cloth dampened with clean water.
- Allow the surfaces to fully dry before using or storing the device.

Suitable tested disinfectants:

Alcohols	Ethanol max. 70%
	Propanol 60%
	Bacillol AF
	Terralin [®] liquid
	Antifect N liquid
	Meliseptol®
Biguanide	Incidin™ Plus
Aldehyde	Kohrsolin Extra
Oxidizing agents	Dismozon® pur
Phenol	Helipur [®]
Activ chlor	Trichlorol®