



MERIVAARA BASIC

Hygienic mattress

Basic hygienic mattress

Merivaara codes 932 / 967 / 934 / 968

Purpose of use

- Merivaara Basic is a good basic mattress for low risk patients in hospitals, nursing homes and home care.
- Weight recommendation for hospital use < 150 kg. If using by a heavy person or 24 hours/day, lifetime is shorter than normally (recommendation in public health care 3-5 years).

Product structure

- Core D33145 (33 kg/m³) fire retardant foam.
- Cover polyurethane coated hygienic bi-elastic fabric, which is impermeable to liquids but still breathable. Material is bi-elastic and treated against bacteria and fungi (active biocidal substance is zinc pyrithione (EU 528/2012). Zipper on two sides.

Benefits

- Secure and easy-care mattress can be used on both sides.
- Facilitate and accelerate the work of staff when cover is easy to clean by wiping.



Cleaning and maintenance

- Disinfection by wiping according to the instructions with diluted chlorine-amine or an alcohol-based detergent or ethanol /chlorine max 1000 ppm, rinse and dry thoroughly. If necessary, wash the cover 70-95°C inside out, gentle tumble drying at low temperature.
- Turn the mattress regularly to ensure even wear. Replace broken cover to a new one.
- The bed frame should be lathed or perforated to allow for proper ventilation.

Flame retardancy & CE

Flammability by standard IMO FTPC part 9. Merivaara Basic mattresses are CE-marked. Country of origin Finland.

Size

Standard sizes 75*200*12, 80*200*12, 85*200*12 and 90*200*12 cm.

