

# manuthera<sup>®</sup> 242

*Master of Spine Techniques*



**Manuthera 242 isn't just any table –**

it's a unique and versatile companion for the most demanding professionals in manual therapy and patient care


**LOJER<sup>®</sup>**

[manuthera242.com](http://manuthera242.com)

# How to utilize Manuthera 242?


## Cervical spine traction

Access lateral aspects of cervical spine joints and ligaments. Treat and relieve tension in the stiff and compressed neck and shoulder region.

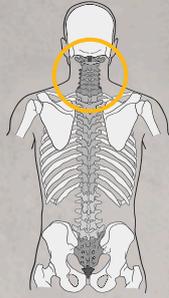


## Thoracic spine manipulation

Manipulate and adjust thoracic spine and ribs for better range of motion and function. Tilting for gravity assistance, flexion and lateral bend from the Manuthera enables more specific and comfortable treatment. Use the dynamic motion for soft tissue techniques and gaining access to deeper tissues.

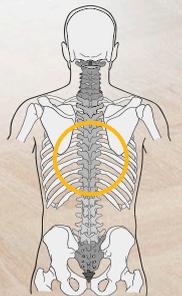

# Lumbar spine


Smooth flexion distraction with side bend, rotation, and decompression options gives you limitless 3D opportunities for lumbar, intervertebral disc, pelvic, and sacrum area treatment. Add some inversion for a broader treatment angle, a stronger effect, treating both superficial and deeper layers of the tissue.

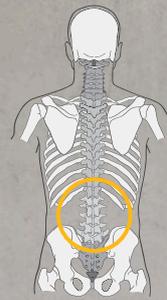


## Discover other possibilities

## Access free videos from global experts!

Access the free video Learning Hub platform by scanning the QR-code below and learn how to get the most out of Manuthera's versatile options. Sign up and you get access to hundreds of videos of different practitioners sharing their expertise with Manuthera 242 and success they've had with various patient cases completely free of charge.

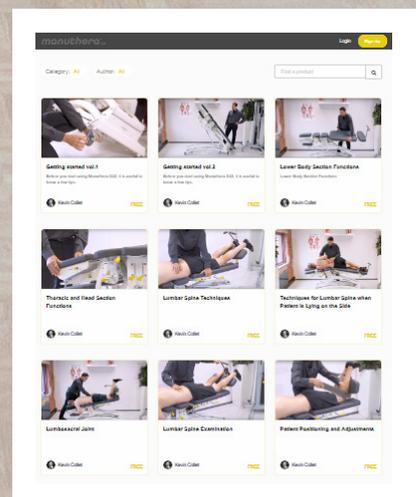

## Manuthera 242 dimensions

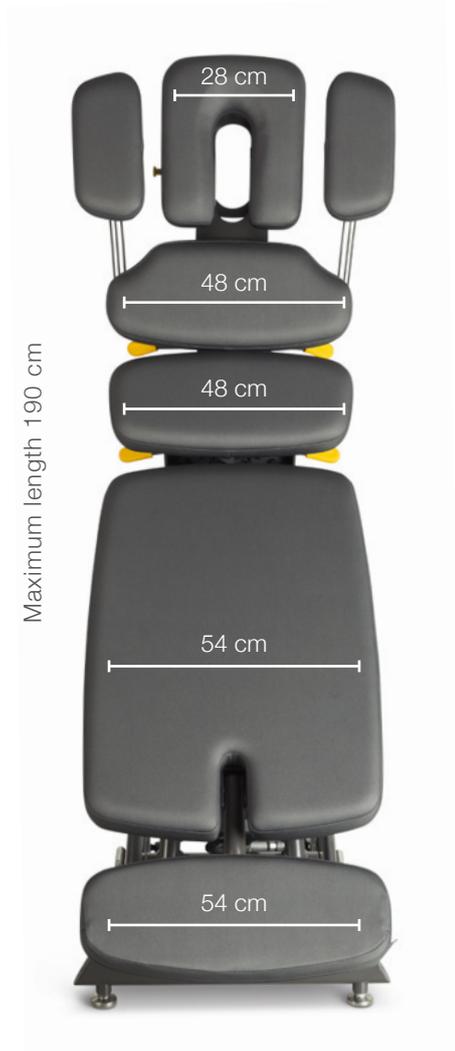

| Technical data and features                    |   |
|--|---|
| Height adjustment                              | 51–106 cm   |
| Weight   | 170 kg  |
| Tilt Up / Down                                 | Electrical 52°/32°  |
| Tilt Up/Down function                          | Electronic  |
| Lower body rotation                            | Yes   |
| Lower body section movement (3D)               | +21.5°/max -40° (vertical), 0..18° (horizontal)                                 |
| Upper thorax -section movement (3D)            | +15°/-13° (vertical), 0..18° (horizontal)                                       |
| Total lock function for upper/lower body       | Yes   |
| Castors  | Yes   |
| Safe working load (SWL)                        | 150 kg  |
| Drop functions                                 | Traction drop   |
| Safety edge                                    | Yes   |
| Safety lock functions                          | Automatic (2 h) or Manual / Manually operated                                   |
| Lying surface                                  | Antimicrobial upholstery  |
| U-shaped face element and breathing hole cover | Yes   |
| Manipulation handle                            | Yes   |
| Country of Origin                              | Finland   |
| CE Marked                                      | Yes The device is a CE Marked medical device according to MDR 2017/745 class 1) |
| Manufacturer                                   | Lojer Oy  |

## FENNIAPRIZE 17

HONOURABLE MENTION

